

# Clearblue

## Ovulation Test DIGITAL

### How Clearblue Digital Ovulation Test works with you

Clearblue Digital Ovulation Test works by detecting the increase (surge) in the hormone called Luteinising Hormone (LH) in your urine. The LH surge occurs approximately 24-36 hours prior to the release of an egg from your ovaries – a process known as **ovulation**. You are at your most fertile on the day your LH surge is detected, **and the day after**.

Your 2 most fertile days begin from the time that Clearblue Digital Ovulation Test detects your LH surge. Make love at any time in the next 48 hours to maximise your chances of becoming pregnant.


### When to start testing

The day that the LH surge occurs varies from woman to woman and from one cycle to another. To have the best chance of detecting your LH surge with the number of Test Sticks provided in this pack, you need to know your usual cycle length. To work out your cycle length, count the day your period starts (first day of full menstrual flow) as Day 1, and continue counting through to the day before your next period starts. The number of days is your cycle length.

If you do not know your usual cycle length or your cycle length varies, refer to question 1 overleaf.


Use the table below to work out the day you should start testing. You can test any time of the day, but it should be at about the same time each day (many women find it most convenient first thing in the morning). It is important that you have not urinated for at least 4 hours before testing, and you should avoid excessive fluid intake before testing.

Length of your cycle (days)	21 or less	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41 or more
Count the first day of your period as Day 1, start testing on the day shown under your cycle length.	5	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	See Question 2 overleaf.

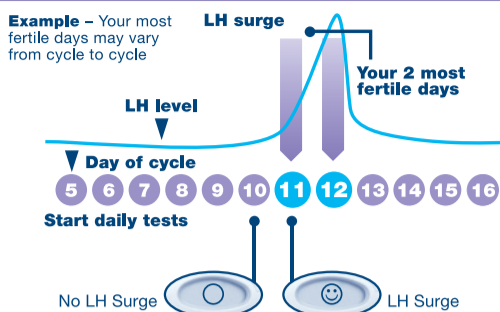

### 1 When you are ready to test...

- Remove the Test Stick from the foil wrapper. Use the Test Stick straight away.
- Before applying urine to the Test Stick it **must** be inserted into the Test Holder. Follow the steps below.

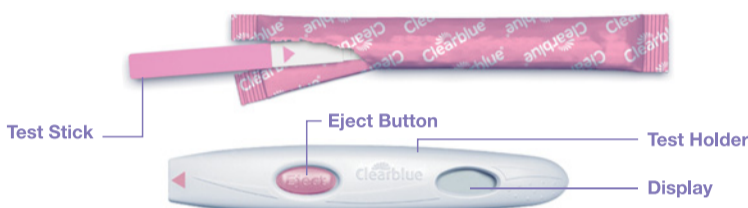


### 2 Remove the Cap

- Find the Pink Arrow on the Test Stick.



### 3 Assemble the Test

- Find the Pink Arrow on the Test Holder.
- Line up both Pink Arrows.
- Insert the Test Stick into the Test Holder until it clicks into place and the Test Ready symbol appears.
- Do not use until the Test Ready symbol appears.
- When the Test Ready symbol appears do the Test straight away.
- If the Test Ready symbol does not appear OR if the symbol has disappeared eject the Test Stick and re-insert it carefully.
- If the symbol still does not appear refer to Question 19 overleaf.

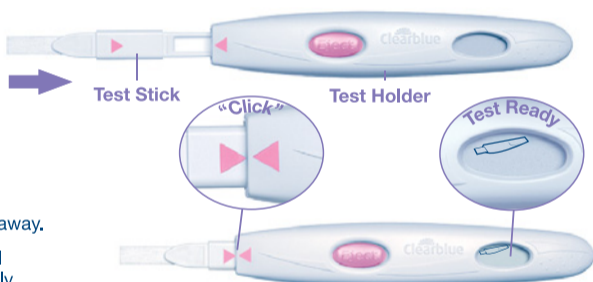

### 4 Do the Test

With the Test Ready symbol showing on the Display...

- Either** test directly in your urine stream.
- Place just the Absorbent Sampler, **pointing downwards** in your urine stream for 5-7 seconds.
- Take care not to get the Test Holder wet.



OR




- Or test a sample of your urine collected in a clean, dry container.
- Dip just the Absorbent Sampler in your collected urine sample for 15 seconds.



### 5 Wait

- Keep the Absorbent Sampler pointing down or lie the Test flat. Throughout testing never hold the Test with the Absorbent Sampler pointing upwards.
- The Test Ready symbol will start flashing after 20-40 seconds to show the Test is working.
- DO NOT EJECT THE TEST STICK.**
- Replace the Cap then wipe off any excess urine.
- If the Test Ready symbol does not start flashing, refer to Question 6 overleaf.

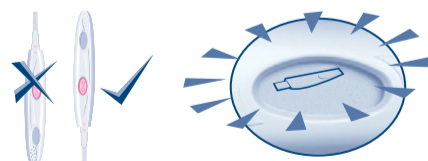

### 6 See your result

- Within 3 minutes the Display will show your result.



- If your result is you have not detected your LH surge. Test again tomorrow at the same time using a new Test Stick.



- If your result is you have detected your LH surge and have reached your most fertile time. Make love at anytime during the next 48 hours to maximise your chances of becoming pregnant. There is no need to carry on testing during this cycle.

- Once you have read your result you **MUST** press the Eject Button and remove the Test Stick.
- The result of your Test can only be read on the Test Holder Display.
- You cannot determine your result from any lines that you see on the Test Stick.

- Your result will remain on the Display for 8 minutes.
- If your Test Stick is still inserted and the Display is now blank, eject the Test Stick and your result will re-appear for 2 minutes.
- Do not re-insert a used Test Stick.

### Error symbols



#### Error A

- You have ejected the Test Stick too soon. Re-insert the Test Stick into the Test Holder immediately. Refer to Question 16 overleaf.



#### Error B

- Refer to Question 17 overleaf.



#### Error C

- Refer to Question 18 overleaf.

## FURTHER INFORMATION

### Disposing of your DIGITAL Test

- Once your result has been displayed, dispose of the test stick with your normal household waste. To do this, press the eject button on the test holder. The test stick will automatically be released. When you eject the test stick you may notice one or two blue lines on the test stick. **These must be ignored**; only refer to the display on the test holder to determine your results. The test stick is not reusable.
- Once you have used all the test sticks in the pack, remove all batteries from the test holder by separating the upper and lower halves starting at the end nearest the display. Remove all batteries from under the central metal cover. Dispose of the batteries according to the appropriate recycling scheme. **Caution:** Do not disassemble, recharge, or dispose of the batteries in fire. Do not swallow. Keep away from children. Dispose of the rest of the holder according to the appropriate recycling scheme for electrical equipment. Do not dispose of electrical equipment in fire.

### Using the Test Holder Again

- Whilst doing a test the Test Holder may have been splashed with some urine. Ensure that you wipe it clean. A further test is not possible whilst your result or any error symbols are on the Display. These will be displayed for 8 minutes (or 2 minutes if your Display was blank and you have just ejected the Test Stick). It is important that you have not urinated for at least 4 hours before doing a test. If you have collected urine you can use the Test Holder again as soon as the Display is blank. When you are ready to use the Test Holder again, repeat the instructions from 1 using a new Test Stick.

## QUESTIONS & ANSWERS

### 1 I do not know my usual cycle length or my cycle length varies. How do I know when to start testing?

If you don't know your usual cycle length you are advised to wait for at least one menstrual cycle, and note the length of this cycle, before using Clearblue Digital Ovulation Test. Once you have determined your cycle length you can use th

2

3

11

12

4

5

6

7

8

9

10

hormonal contraception and if you have recently stopped using it your cycles can be irregular and may take some time to stabilise. You may wish to wait until you have had two natural menstrual cycles, and note the length of these cycles, before using Clearblue Digital Ovulation Test.

### 13 Since it tells me when I am fertile, can I use Clearblue Digital Ovulation Test as a contraceptive?

No. Clearblue Digital Ovulation Test detects the LH surge, which occurs only 24-36 hours before ovulation. As sperm can survive for several days, you might still become pregnant if you made love before you detected your LH surge.

### 14 I think I have become pregnant, what should I do?

You can find out if you are pregnant up to 4 days before your expected period using a Clearblue Digital Pregnancy Test with Conception Indicator. If your result is 'Pregnant' the test will also indicate how many weeks ago you conceived. In laboratory testing Clearblue Digital Pregnancy Test with Conception Indicator is more than 99% accurate in detecting whether or not you are pregnant when used from the day your period is due. If you are pregnant you should see your doctor who can advise what steps to take next. The earlier you see your doctor, the better it is for your own health.

### 15 I've used Clearblue Digital Ovulation Test for several months and haven't become pregnant. Can I be sure of getting pregnant?

It can take many months to become pregnant. There are many reasons why you may not become pregnant even if you've been able to make love at your most fertile time. If after several months you've had no success, you should consult your doctor.

## Error Messages

### 16 A flashing symbol (Error A) has appeared on the Display. What does this mean?



The Test Stick has been ejected too soon. If it is not re-inserted quickly enough then a further error symbol will appear on the Display. Refer to question 17.

### 17 This symbol (Error B) has appeared on the Display. What does it mean?

An error has occurred during testing. It may be that

- Urine has been applied to the Absorbent Sampler before the Test Stick was inserted into the Test Holder.
- The Absorbent Sampler was not kept pointing downwards, or the Test was not laid flat, after urine was applied.
- Too much or too little urine was applied.
- The Test Stick was ejected too soon.

This symbol will be displayed for 8 minutes. You should test again using a new Test Stick taking care to follow the instructions. Refer to 'Using the Test Holder again'.

### 18 This symbol (Error C) has appeared on the display. What does this mean?



The test has not performed as expected. This may be because the instructions for use were not followed. If you have unused test sticks in the pack, **do not** use them with this holder. You will need to perform another test from a new pack. Please follow the instructions for use carefully.

### 19 The Display has remained blank. What does this mean?



If you have not applied urine to the Test Stick eject it and start again from step 3. Otherwise, please contact the Clearblue Careline.

[www.clearblue.com](http://www.clearblue.com)

Advisors available 07:00 – 15:00hrs Mon-Fri, excluding Bank Holidays.

**UK 0800 917 2710 • IE 1800 812 607**

Free from landlines, mobiles may be charged. All calls are recorded and may be monitored for quality purposes. Please be ready to quote the number.

As prenatal care is very important for a baby's health, we recommend that you consult your doctor before trying to get pregnant.