



Fertility Friendly Lubricant


For use when trying for a baby

Unlike ordinary lubricants which can harm sperm, Clearblue Fertility Friendly Lubricant has been designed to create the optimal environment for sperm survival, which is why it is most suitable for use by couples trying to conceive. By working in harmony with your body it provides comfort and lubrication during sex.

Clearblue Fertility Friendly Lubricant is pH balanced and isotonic to be sperm friendly. The light texture helps maintain sperm health, survival and movement and comes in a pre-filled tube for easy application.

If you're trying for a baby have you thought about using an ovulation test? Clearblue offers a wide range of ovulation tests that identify your fertile days to maximise your chances of getting pregnant naturally.

1 When to use

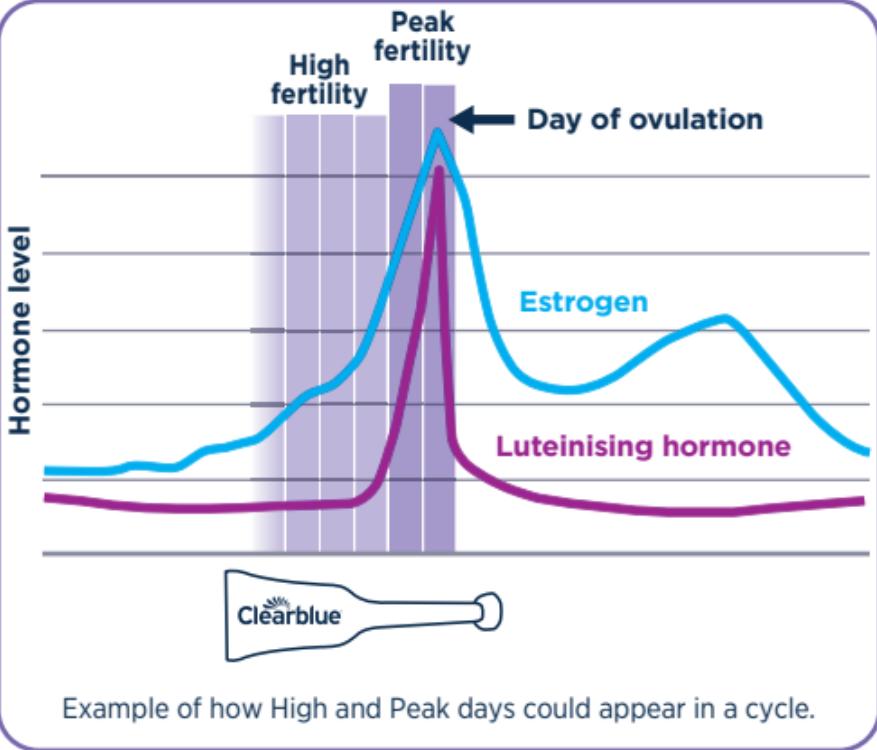


Use one applicator before having sex.
The lubricant is designed for couples trying to conceive so for best results use during your most fertile days. Insert a maximum of 15 minutes before having sex.

2


How to use

You may find inserting the tube easier when laying down.

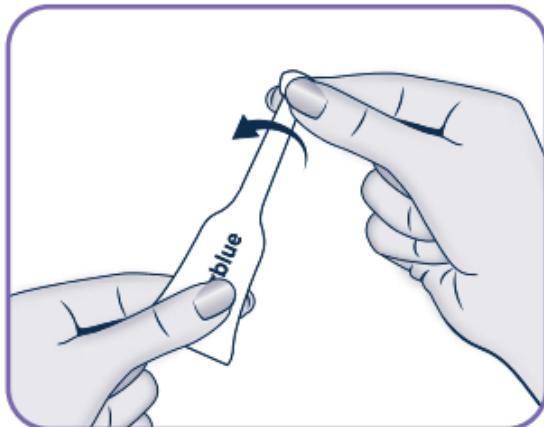

Take one pre-filled tube and hold so that the contents move towards the narrow end of the tube.

Twist off the tab at the narrow end of the tube.

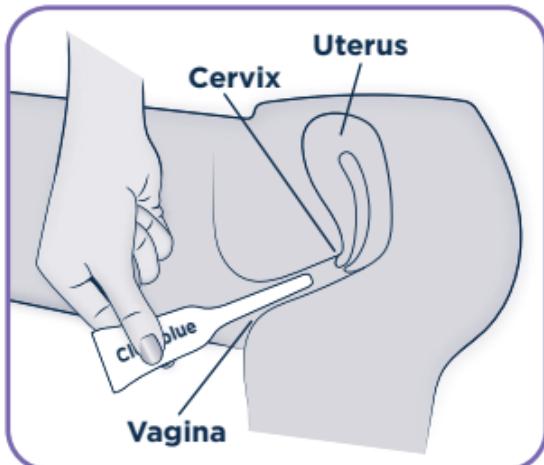

Before having sex, insert the narrow end of the tube into your vagina.

Squeeze out the contents.

Keep squeezing as you remove the tube.

Discard the tube.



www.clearblue.com

Advisors available 07:00 - 15:00hrs Mon-Fri,
excluding Bank Holidays.

UK 0800 917 2710 • IE 1800 812 607

Freephone.

Calls are recorded and may be monitored for quality purposes. Please have the product and packaging with you when you call.

There is no guarantee of pregnancy when using Clearblue Fertility Friendly Lubricant. No patient data is available regarding viable pregnancies or birth outcomes in patients using this product.

Read the enclosed leaflet before use. Single use only.

Do not use if the tube is damaged. If irritation or discomfort occurs, discontinue use and see a doctor. Keep out of reach of children. This is not a spermicide or a contraceptive. Do not use if you are allergic to any ingredients. Do not use if tube has passed its expiry date. Each tube is for single use only and cannot be stored after opening or reused.

Contents: 7 x 4ml applicators

Ingredients: Water, Phosphate Buffered Saline, Sodium Hyaluronate (Hyaluronic Acid), Phenoxyethanol, Methylparaben

Storage: store between 5 - 25°C

	Manufacturer		Do not re-use
	Batch code		Consult instructions for use
	Use-by date		Temperature limit 5°C - 25°C
	Material recycling		

Rolf Kullgren AB, Ågatan 4, 646 30 Gnesta, Sweden
Clearblue is a trade mark of SPD.

© 2021 SPD. All rights reserved.

2300620.01 02-2021 14106-01

